

LINQ™ FAMILY OF ICMS



Reveal LINQ™ ICM¹



LINQ II™ ICM²

DEVICE CHARACTERISTICS

Parameter	Reveal LINQ™ ICM ¹ Value	LINQ II™ ICM ² Value
Longevity	3 years	4.5 years*
Volume	1.2 cc	1.4 cc
Mass	2.5 g	3.4 g
Electrode Spacing	38 mm	40 mm
MRI Compatibility	1.5 and 3T	1.5 and 3T
Telemetry	Inductive (Tel B) One-directional RF (MEDS)	BlueSync™ technology (Bluetooth® Low Energy, Two-way communication)

COMPATIBLE DEVICES

In-clinic Programmer	CareLink™ 2090 Programmer, or CareLink Encore™ 29901 programmer, or Reveal LINQ™ Mobile Manager	Reveal LINQ™ Mobile Manager
CareLink™ Network	Yes	Yes
Patient Monitor	MyCareLink™ Patient Monitor	MyCareLink Heart™ Mobile App, or MyCareLink Relay™ Home Communicator
Patient Symptom Marker	Patient Assistant (PA96000)	MyCareLink Heart™ Mobile App, or Patient Assistant (PA97000, for MyCareLink™ Relay only)

PROGRAMMING CAPABILITIES

In-clinic programming mode	Non-wireless programming	Wireless programming
Remote Programming	Not available	Available via Carelink™

DETECTION CAPABILITIES

Detection Algorithms	TruRhythm™ Detection <ul style="list-style-type: none"> ▪ Pause Detection ▪ Tachy Detection ▪ Brady Detection ▪ AF Detection 	Enhanced TruRhythm™ Detection <ul style="list-style-type: none"> ▪ Pause Detection Algorithm: enhanced smart filter, noise rejector, loss of contact rejector ▪ Tachy Enhancement: Rapid Onset ▪ Brady Detection Enhancement: Nighttime Storage ▪ AF Detection ▪ PVC Detection
Patient Settings	Initial programming defaults, based on Reason for Monitoring	Enhancement to programming defaults, based on Reason for Monitoring

ECG DATA STORAGE AND TRANSMISSION

	Reveal LINQ™ ICM ¹	LINQ II™ ICM ²
Parameter	Value	Value
Smart Memory Management	No	Yes Up to 23 full ECG pr day stored & transmitted (no need for manual transmission)
Clinician Notification	Nightly transmission CareAlert™ notifications	Daily/Hourly transmission depending on the monitor. CareAlert™ notifications between 5–6 a.m. clinic time
Alerts Settings	Programmable CareAlerts	Automatic CareAlerts, based on Reason for Monitoring
Episodes Storage	59 min	61 min
Cardiac Compass™	Yes	Yes Enhanced with: ▪ Patient Symptoms ▪ PVC burden
Remote Programming	No	Yes

CLINIC AND PATIENT SERVICES

BeConnected Patient Service	Patient Education	Patient Education, Remote Monitoring Onboarding and Monitor Setup
FocusOn™ Monitoring & Triaging Service	Yes	Yes

¹ Reveal LINQ LNQ11 ICM Clinician Manual M958488A001 Rev D.

² LINQ II LNQ22 ICM Clinician Manual. M974764A001D.

*Nominal settings.

Brief Statement

See the device manual for detailed information regarding the instructions for use, the implant procedure, indications, contraindications, warnings, precautions, and potential adverse events. If using an MRI SureScan™ device, see the MRI SureScan™ technical manual before performing an MRI. For further information, contact your local Medtronic representative and/or consult the Medtronic website at www.medtronic.eu.

For applicable products, consult instructions for use on www.manuals.medtronic.com. Manuals can be viewed using a current version of any major internet browser. For best results, use Adobe Acrobat® Reader with the browser.

Medtronic and the Medtronic logo are trademarks of Medtronic. The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Medtronic is under license. Other third party brands are trademarks of their respective owners. All other brands are trademarks of a Medtronic company.

Medtronic

Europe
Medtronic International
Trading Sàrl.
Route du Molliau 31
Case postale
CH-1131 Tolochenaz
Tel. +41 (0)21 802 70 00
Fax +41 (0)21 802 79 00

medtronic.eu

United Kingdom/Ireland
Medtronic Limited
Building 9
Croxley Park
Hatters Lane
Watford
Herts WD18 8WW
www.medtronic.co.uk
Tel: +44 (0)1923 212213
Fax: +44 (0)1923 241004

UC202100945EE © Medtronic 2021.
All rights reserved.